MC Care Liquid Deutsch

Gebruichsanweisung

Produktbeschreibung

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Operating Instructions

Product description

MC Care Liquid (REF 66 31 191) is a preventative care and cleaning concentrate for use in the wet cleaning procedure of all Dentsply Sirona grinding and milling machines. In addition to the cleaning function, the liquid provides a maintenance task. It produces a pearling effect and has a slight decalcifying effect. If MC Care Liquid is used with each wet cleaning process, contamination in the processing chamber, hose system and nozzles is greatly reduced.

Intended use

MC Care Liquid may only be used for cleaning the processing chamber and water system of Dentsply Sirona grinding and milling machines. It must generally be diluted (1ml concentrate : 200ml water).

Application procedure

The MC Care Liquid shall be used as follows:

1. Drain water tank and remove filter.
2. Thoroughly rinse water tank with water.
3. Clean or, if necessary, replace water filter.
4. Fill water tank to a normal level until the filter is completely covered with water.

Water quantity to the lower edge of the cover thread:

- CEREC MC / MC X / MC XL / MC XL Premium Package / Primemill and inLab MC XL: approx. 3 liters
- inLab MC X5: approx. 10 liters

5. Add MC Care Liquid to the water tank:

- CEREC MC / MC X / MC XL / MC XL Premium Package / Primemill and inLab MC XL: 15ml or 1x up to **MC XL** marking
- inLab MC X5: 50ml or 2x up to **MC X5** marking

6. Close the water tank, to do so tighten the tank cover by hand in a clockwise motion.
7. Reconnect the water tank to the device.
8. Switch on the wet cleaning process:
 - CEREC Primemill: Select „*Routine actions*“ and then „*Cleaning program*“. There are two cleaning options: 2 minutes and 15 minutes. Each cleaning option can be stopped at any time when the desired results have been achieved.

- CEREC MC / MC X / MC XL / MC XL Premium Package and inLab MC XL: In the software system menu, select the buttons „*Configuration*“ and then „*Devices*“. Select the grinding unit to be cleaned here. Then start the cleaning process under „*Configuration*“ and then „*Service*“.

- inLab MC X5: Perform an automatic cleaning process (without a clamped round) via the service menu of the inLab CAM software.

9. Repeat the wet cleaning cycle, if necessary.
10. Completely drain the water tank with MC Care Liquid and rinse with water.
11. „Connect an empty water tank to the device and empty the hose system of the device by pressing the * button.
 - * „Water pump“ for Primemill,
 - * „Pump“ for MC/MC X/MC XL/MC X5
12. Empty the water tank again and rinse it out.
13. Refill the water tank to a normal level and reconnect it to the device.

Notes

MC Care Liquid is not provided for cleaning heavily soiled devices. Persistent dirt can be removed with undiluted MC Care Liquid on a paper towel.

In the case of processing restorations, MC Care Liquid may not be mixed into the water and Dentatec solution.

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