


User Manual

Attention: Please read this manual before you use it.

A9W 5G WIFI Dental Camera

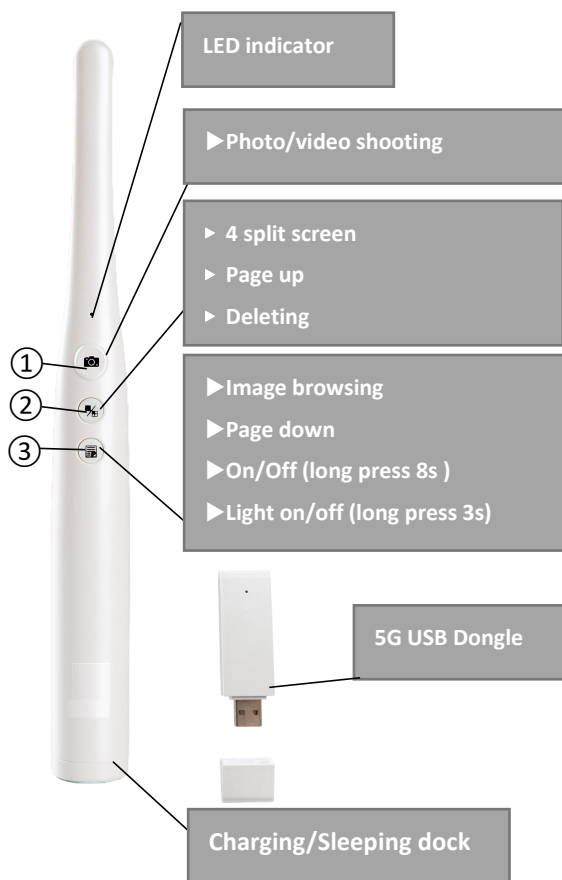

Packing list:

- Handheld oral camera * 1
- USB dongle * 1
- Charging dock *1
- Micro USB charging cable * 1
- Protection sheath * 50

Product Introduction:

This product is mainly composed of handheld oral camera, USB Dongle and charging dock etc. It is a micro photography system specially designed for dentists. It helps dentists to conduct oral examination, diagnosis and image shooting.

Technical Specification:

Model name: A9W
Graphic sensor: 1/6 inch CMOS sensor;
Camera: 2MP Macro lens;
Focus distance: 20mm-50mm
Fill in light: 12 led 5500k color temperature
Output image:1280*720 JPG format
Battery: 3.7V 2500mAh li-ion battery
Working time: 8+ hours constant work (full charge)
Transmission distance: 10m
Working Temperature: 5-40℃ / 41-104°F
Working humidity: ≤80%
Working atmospheric pressure: 86.0kpa-106.0kpa
Charging Power: 5V/2A (power adapter not included)
Storage: put in dry package and avoid direct sunlight;

Software Installation on windows PC:

- 1,Install Uteeth software on windows PC;
- 2,Insert USB dongle to USB port of the windows PC;
- 3, Press button ③, you will notice the LED indicator turns blue, it means the machine is turning on. The dental camera will connect to dongle automatically. If camera failed to connect to dongle, then please short touch button ① and the LED indicator will blink fast, which means it's connecting to USB dongle.
- 4, After connecting, you will be able to see real-time image on PC software(Uteeth).

Operation of oral camera:(working with Uteeth)

❖ Photo/Video shooting

Short touch button ① to shoot photos;
Long touch button ① to shoot videos;

❖ 4 split screen

At single image taking status,short press button ② and you will enter 4 split screen mode; Press ① to freeze image,press ③ to refreeze image; When 4 images frozen, touch button ① to store image and start new 4 split screen image taking mode. Press button 2 to exit 4 split screen mode.

❖ Photo browsing

When at photo shooting mode, you press button ③ to enter photo browsing mode. You can shift images by pressing button ② for page up or pressing button ③ for page down. Short press button ① to exit browsing mode.

❖ Video browsing

You can see video list menu at left side of Uteeth software on PC. Double click the video file you choose to play it.

❖ Delete photos

When at photo browsing mode,short press ② or ③ to choose file, and long press ② 3 more seconds for deleting, and short press ② to finally confirm deleting.

❖ Turn on/off LED

When at photo shooting mode, long press ③ 3 seconds to turn on or off LED.

❖ Turn on/off LED

When you take oral camera out of charging dock, it will automatically turn on; and when you put it back to charge, it will automatically enter sleeping mode;
You can long press button ③ for 8 more seconds(LED light off) to turn off the device manually.