

SmartLite® Focus®
Pen-Style LED Curing Light

Focused
on Essentials




THE DENTAL
SOLUTIONS
COMPANY™

 Dentsply
Sirona

Focused on Essentials


- Reliable Curing
- Convenient Clinical Handling
- Virtually Permanent Availability

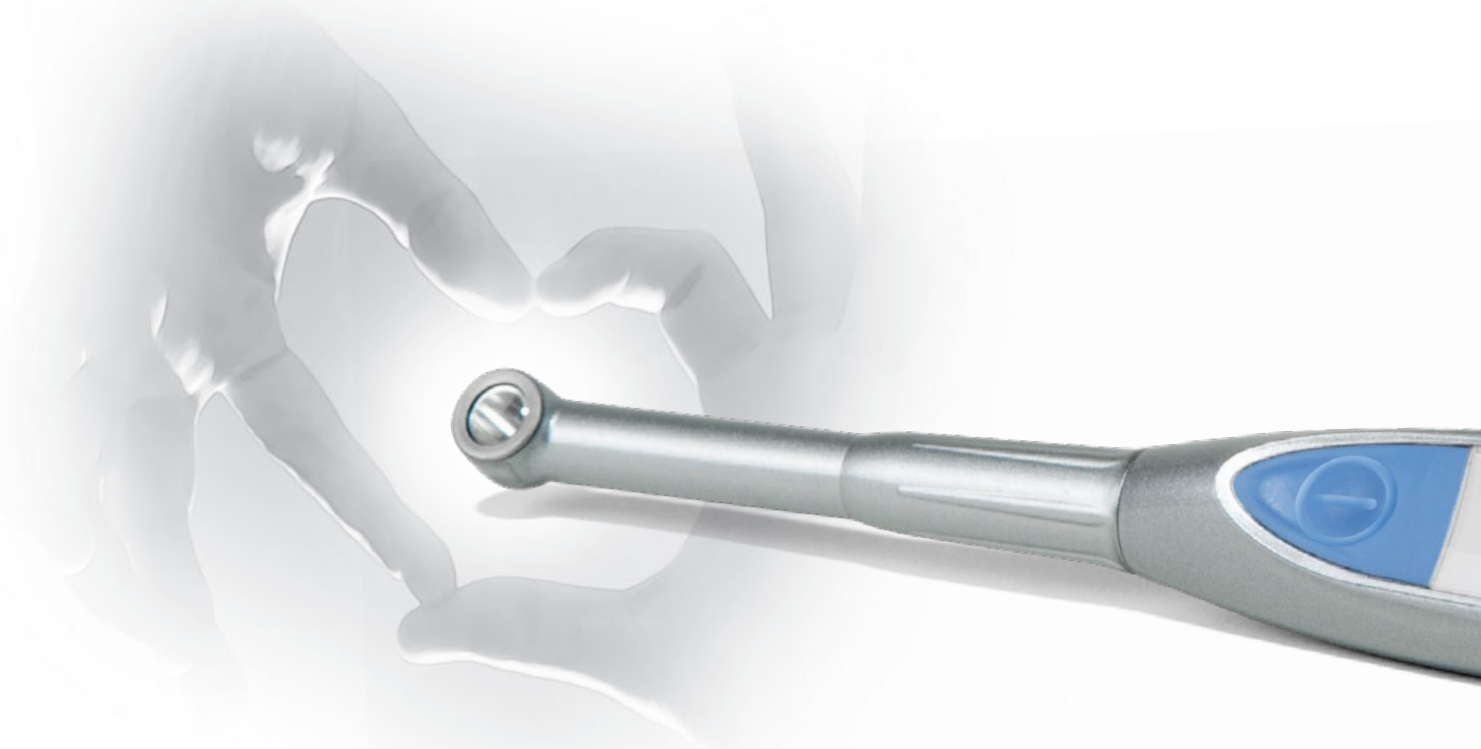

Focused on Reliable Curing

The SmartLite® Focus® optics reduces divergency and ensures a homogeneous beam profile.

This ensures the recommended depth of cure¹ for most Dentsply Sirona light curing restoratives even up to 8 mm distance^{2,3}.

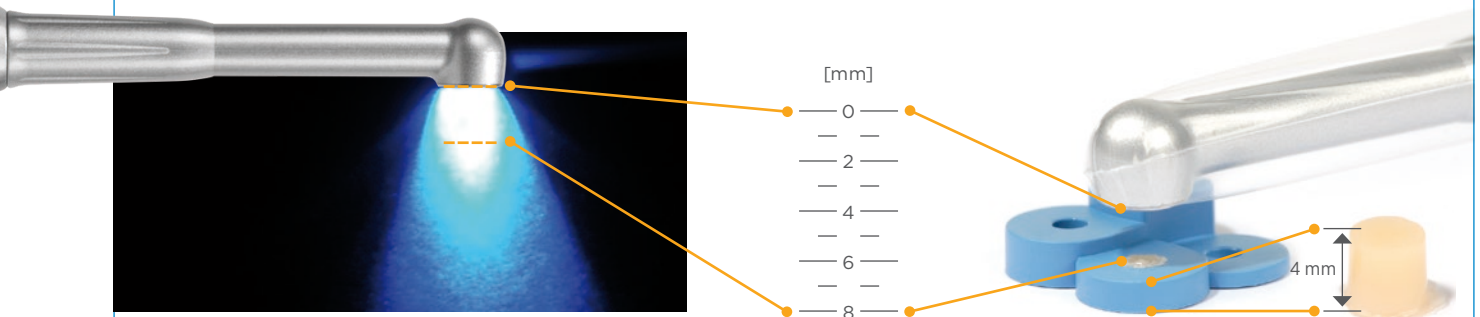

The clinical benefit

- Reliable light curing over larger distances
- Homogeneous performance within the curing area

Ceram-X® mono+ M5 cured at 8 mm distance, using the i-cure (curing depth device).

4 mm of cured material = 2 mm depth of cure according to ISO.

¹ According to SmartLite Focus Curing Guide for Dentsply Sirona Restorative Materials.

² Depth of cure measured according to ISO 4049; Data on file.


³ Curing Light shall be positioned as close as possible to the restorative material in the given clinical situation.

Focused on Access and Clinical Handling

SmartLite® Focus® provides a slim pen-style design with a long, rotatable tip and a one-button control, activating a 20 sec. curing cycle.

The clinical benefit

- Excellent Intra-oral access
- Easy one-touch handling
- One mode for direct and indirect restorations

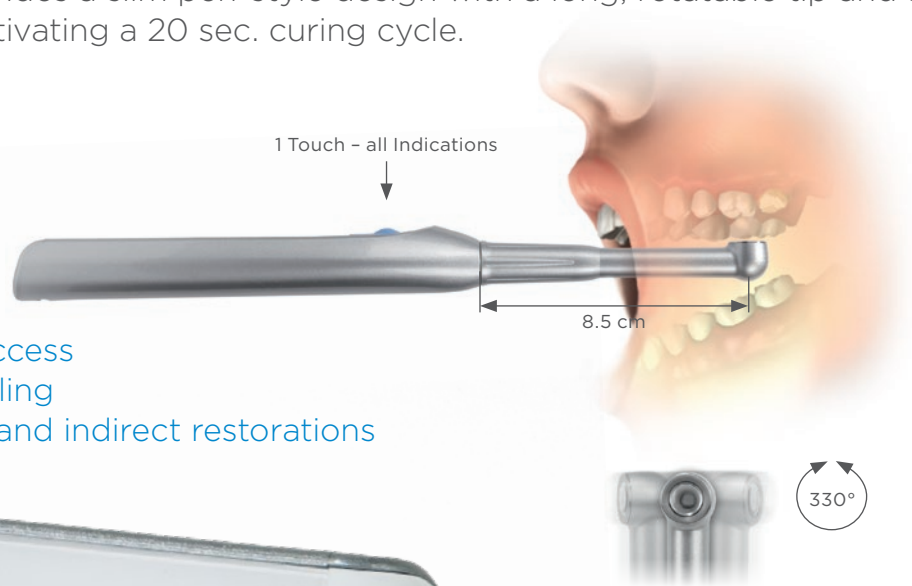

Focused on Availability

SmartLite® Focus® is instantly ready to use upon purchase without need of initial charging. The SmartLite® Replaceable Battery Pack offers stable energy output even after prolonged storage. The SmartLite® Focus® “Smart Recharge Technology” checks the curing light’s battery conditions and initiates the appropriate charging mode:

“Automatic Quick Charging Mode”

Provides at least 5 min. curing time after 10 min. charging when battery is nearly empty

“Standard Charging Mode”

Regains full capacity in less than 3 hrs.

“Standby Mode”

Prevents overcharging upon reaching full capacity

The clinical benefit

- Virtually permanent availability



Light Beam Testing Area



Experience SmartLite® Focus® and perceive the collimated light beam and the minimal diffusion.

Ordering Information

SmartLite® Focus® Intro Kit 644.50.020

1 x SmartLite® Focus®
1 x SmartLite® Focus® Charging Base
1 x SmartLite® Focus® Battery Pack
2 x SmartLite® Focus® Shields
50 x SmartLite® Sleeves
1 x SmartLite® Focus® Power Connector
1 x Instructions for Use
1 x Material Curing Guide

SmartLite® Focus® Charging Base Refill 644.50.070

1 x SmartLite® Focus® Charging Base

SmartLite® Focus® Battery Refill 644.50.021

1 x SmartLite® Focus® Battery Pack

SmartLite® Sleeves Refill 644.50.030

300 x SmartLite® Sleeves

SmartLite® Focus® Shields Refill 644.50.040

5 x SmartLite® Focus® Shields

SmartLite® Focus® Power Connector Refill 644.50.050

1 x SmartLite® Focus® Power Connector + Adapters



Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
www.dentsplysirona.com
Phone +49 (0)7531 583-0

Dentsply Limited
Building 3
The Heights, Brooklands
Weybridge, Surrey KT13 ONY
UNITED KINGDOM
Phone +44 (0)800 072 3313

K79200077-06
© Dentsply Sirona, Inc. 6/2018