


# EASY360 PLUS MOBILE PHONE ADAPTER

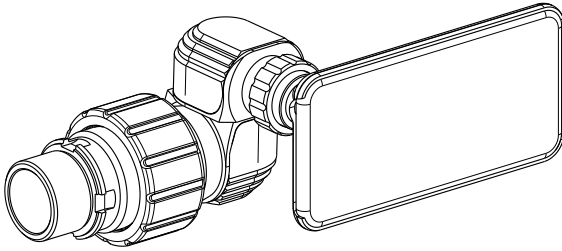

## User's Guide



**CE** The CE mark indicates that the product complies with the European medical device regulation (EU) 2017/745.

Thank you for purchasing the Easy360 Plus Mobile phone adapter. To prevent damage to the Easy360 Plus Mobile phone adapter or injury to you or others, read the following safety precaution in their entirety before using this equipment. Keep these safety instructions where all those who use the device will read them.

### Cautions

1. Read and follow all caution notices and instructions marked on the product or included in the documentation.
2. Do not use this instrument in damp rooms. Do not expose the instrument to water splashes, dripping water or sprayed water.
3. Do not touch the surface of the lens with hands or hard objects..
4. Do not operate the knob because it is easy to damage the product.
5. This product uses a mobile phone set only match the corresponding smart phone.

### 1. General instructions

#### 1.1 Purpose

For medical observation or teaching use, must be used with Zumax surgical microscope, and the surgical microscope should be equipped with the corresponding light splitting system.

#### 1.2 Characteristic

- Easy360 Plus Mobile phone adapter can be directly equipped with smart phones to take pictures;
- Various types of mobile phone card sets to adapt to a variety of models of smart phones;
- Adjust the image knob can adjust the direction of the image, so as to facilitate the user to observe;
- Easy360 Plus Mobile phone adapter connector has 4 ways to install;
- Easy360 Plus Mobile phone adapter connector can be equipped with Wireless footswitch of Easy360 Plus Mobile phone adapter.

### 2. Installation

#### 2.1 Easy360 Plus Mobile phone adapter installation.

2.1.1 Install mobile phone to the phone set and ensure that the phone is firmly installed.(Figure 2.1).

2.1.2 Install the mobile phone set to the phone adapter body, fix the direction of the mobile phone, tighten the set ring to install and fix the mobile phone set.(Figure 2.2)

2.1.3 loosen the set ring can adjust the direction of mobile phone, mobile phone can be fixed after the angle adjustment screw the set ring is fixed phone sets.

(Figure 2.3)

2.1.4 The direction has two axial direction adjustable.

(Figure 2.4)

2.1.5 Install the phone camera adapter into the light splitting system, and the corresponding optical connector interface has 4 mounting directions. The phone camera adapter can be fixed by pressing joint ring. (Figure 2.5)

2.1.5 Start the phone camera app, adjust the knob can adjust the camera image direction.(Figure 2.6)

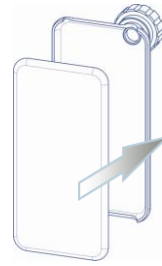


Figure 2.1

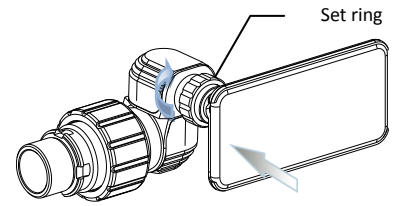


Figure 2.2

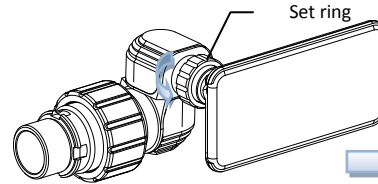


Figure 2.3

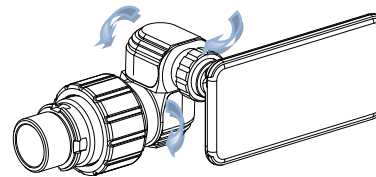


Figure 2.4

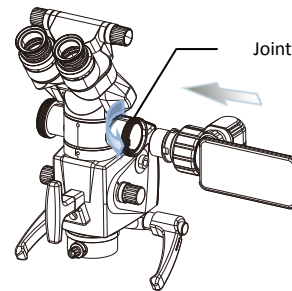


Figure 2.5

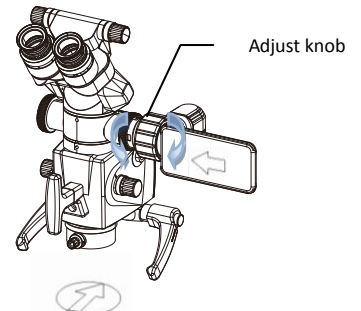


Figure 2.6

### 3.1 Cleaning

Clean with a damp cotton swab with warm water and mild detergent.

### 3.2 Maintenance

Servicing is required when the device has been damaged in any way, or have any questions, please contact the company authorized dealer or direct contact with the company.



**Zumax Medical Co., Ltd.**

5 Zhiying Street, Suzhou New District,

Suzhou, Jiangsu, China 215129

Phone: +86 512 66650502

Fax: +86 512 66909655

E-mail: Sales@zumaxmedical.com

Http://www.zumaxmedical.com



MedNet EC-REP GmbH

Borkstrasse 10, 48163 Münster, Germany

编制	杜磊	日期	20210426						手机显微影像系统 Easy360 Plus
校对		日期							
审核		日期							
批准		日期							使用说明书
归档		日期		标记	处数	更改文件号	签字	日期	
归档介质			版本号	苏州速迈医学科技股份有限公司				85292. 200. 00. E3	
文件页数			3						
注：本页本栏仅作为归档时使用。									