




Table of Contents

BEYOND® II Whitening Accelerator Product Overview	«01
Turning on Your BEYOND® II Whitening Accelerator	«04
Touch User Interface (TUI) Control Panel	«05
Simultaneous/Dual Whitening Option	«06
Light Output Settings	«07
Whitening Treatment	«07
Air Ionizer Purifier	«10
Whitening Gel Storage and Maintenance	«10
Safety Measures	«11
Troubleshooting	«11
Manufacturer's Contact Information	«12

Thank you for your purchase. BEYOND® International Inc., global leaders in aesthetic dental products is proud to provide you the BEYOND® II Whitening Accelerator which will allow you to provide superior whitening results at an affordable price.

Purpose

The BEYOND® II Whitening Accelerator is a reinvented version of our BEYOND® Power Whitening System designed for professional whitening in a dental facility to whiten stained teeth. With the BEYOND® II, a high-intensity LED light is used to deliver a tailored beam of light between 480 – 520 nanometers in wavelength needed to activate BEYOND® whitening gels and provide up to 6-8 shades improvements.

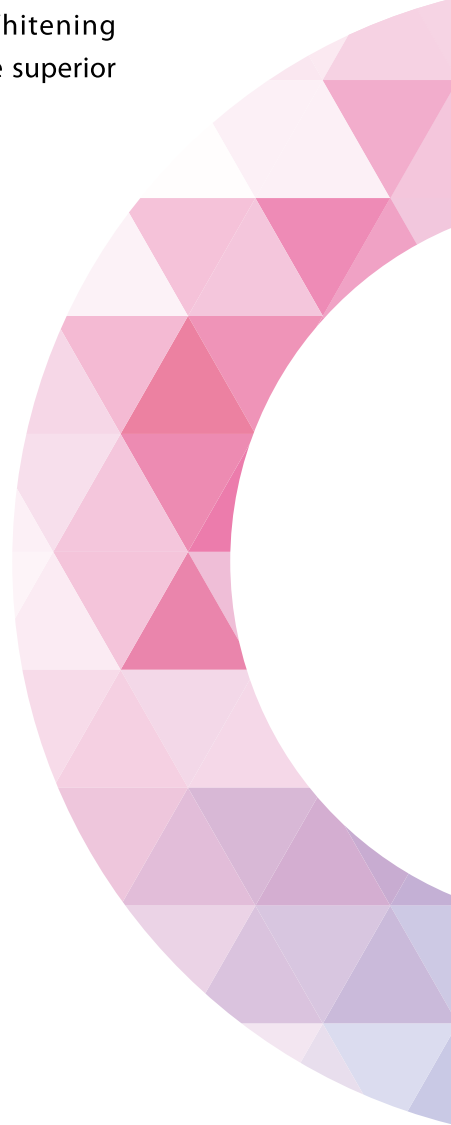

beyond[®] II Whitening Accelerator

01
02



Included in the BEYOND[®] Package

- BEYOND[®] II Whitening Accelerator
- BEYOND[®] II LED light
- BEYOND[®] II cheek retractors(10)
- Power Cord
- Protective eye goggles
- Operational manual
- Wall posters(2)
- Four-fold promotional brochures (100)

All BEYOND[®] brand dental products are manufactured to international quality standards and are thoroughly inspected upon completion. Once received, inspect both the shipping packaging and the whitening system for any damage and report it, if any, to BEYOND[®].

Features

Elegant – Chic and modern design embraced by any dental practice.

Simultaneous – Allows for the effective treatment of two patients at the same time.*


Unique – Advanced design allows for a portable system, allowing for ease of mobility through different rooms within the practice.

Adjustable – Whitening option can be set to high, medium, or low light output settings for a tailored treatment that best suits each individual patient.

Digital – Simple and quick patient preparation and system operation with digital display control board.

Purifies – Using negative ions, the built-in air purifier reduces airborne pollutants that may exist within the practice, actively contributing to a cleaner and healthier environment.

*Second LED sold separately.

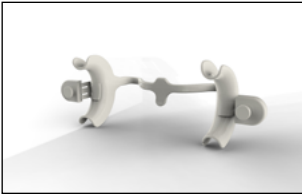

Technical Data

Input Voltage: 100~240VAC, 50/60Hz

Output Voltage: DC 24V

Input Power: 40VA

Rated Current: 360mA

Illuminance:

High: 26000LX

Medium: 23000LX

Low: 20000LX

Wave Length: 485nm

Light Working Power: 40mW/cm²

Light Working Voltage: DC 24V

Fuse type: T1.0 L250V

Gross Weight: 6KG


Package Dimension: 81 x 32.5 x 23cm

Recommend Treatment Time: 12min. x 3times


Turning on Your BEYOND® II Whitening Accelerator



After checking the voltage input setting, plug the machine into a grounded power unit and switch the POWER ON/OFF switch located at the back of the machine near the power cable plug to ON.

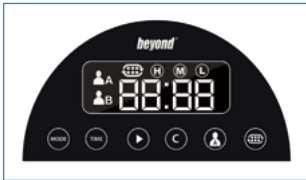

The TUI panel will illuminate the default time on the control panel. Your Beyond II Whitening Accelerator is now ready to be used.

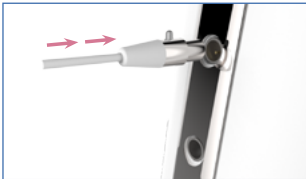

Up to two BEYOND® II LED lights may be connected to the BEYOND® II Whitening Accelerator. There are two male 3-pin circular connector sockets at the front of the BEYOND® II Whitening Accelerator. Push the female 3-pin circular connector attached to the BEYOND® II LED light until it stops.




To remove the BEYOND® LED light, press the black button on the circular connector, which allows the release of the BEYOND® LED light.




To turn off the BEYOND® II Whitening Accelerator, all procedures should be reset. Then switch the POWER ON/OFF switch located at the back of the machine to OFF.


Touch User Interface (TUI) Control Panel





[A] **High / Medium / Low:** Select light intensity by pressing the MODE  button. The intensity levels may be changed before the desired procedure time has started or after the procedure is completed. The chosen intensity level will be lit next to the treatment cycle timer.

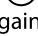
[B] **TIME:** Press to adjust the treatment cycle time. Factory default is set at 12 minutes. Longer procedure times may be needed for more severe cases. The system will default to the last time selected. Procedure time may not be adjusted during the procedure.

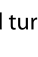[C] **START/PAUSE:** Press the  to turn on accelerator light and begin countdown of treatment cycle. Press the  again to pause the treatment. Press  to resume treatment.

[D] **CANCEL:** Press  to stop the procedure and turn off accelerator light. This will reset time to 12 minutes.

[E] **PA/PB:** Press  to operate the functions on either Patient A or Patient B. The chosen PATIENT function that will be operable will be lit next to the treatment cycle timer.

[F] **AIR PURIFIER:** Press  to turn on air purifier on. The air purifier icon will be lit next to the treatment cycle timer when operable.

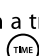[G] **TREATMENT CYCLE TIMER:** Shows the minutes remaining in a treatment cycle. Factory default is 12 minutes although this can be adjusted using the TIME  button.

Simultaneous / Dual Whitening Option


The BEYOND® II Whitening Accelerator gives you the option to whiten two patients simultaneously. Although the BEYOND® II Whitening Accelerator includes one BEYOND® II LED light, a second LED light may be purchased to offer the simultaneous option.




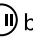




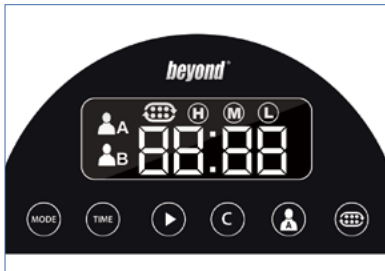

05
06





Plug in the two BEYOND® II LED lights to the 3-pin circular connector sockets. Top socket is for Patient A LED light function. Bottom socket is for Patient B LED light function



Press PA/PB  button to pick which LED light to function. Press TIME  button to set the time for the procedure. Press the MODE  button to choose the appropriate light intensity for the patient. Press START/PAUSE  button to turn on the BEYOND® II LED Light, starting the whitening procedure.

Press the PA/PB  button to switch to the other patient. Press TIME  button to set the time for the procedure. Press the MODE  button to choose the appropriate light intensity for the patient. Press START/PAUSE  button to turn on the BEYOND® II LED Light, starting the whitening procedure.

You may pause the procedure for each of the two patients during the procedure by pressing the PA/PB  button to choose the appropriate patient and then press then START/PAUSE  button.

Light Output Settings

LOW

The light output for this setting is 20000LX and is recommended for patients with sensitive teeth, severe fluorosis, enamel hypoplasia, or visibly compromised enamel (small cracks or chips). When using this setting, treatment times may need to be extended to account for weaker light output levels.



MEDIUM

The light output for this setting is 23000LX. This setting is recommended for patients with healthy enamel, yellow stains caused by consumption of staining foods and drinks, and tobacco use. Patients with mild fluorosis can be treated using this light level, as well as mild tetracycline stains.

HIGH

The light output for this setting is 26000LX and is recommended for patients with severe tetracycline stains, and gray, or related discolorations.


Note: A high percentage of patients with severe tetracycline stains may experience sensitivity. Patients with healthy, thick enamel and extrinsic, yellow discolorations can also use this setting if dramatic whitening results are desired.

Whitening Treatment


The treatment procedure outlined in this manual is the standard BEYOND® Whitening Treatment as recommended for patients with common extrinsic stains caused by staining foods, drinks, and tobacco products. Recommendations on treating severe extrinsic and intrinsic stains (including tetracycline and fluoride stains) can be found on our website at www.beyonddent.com/treatments or in the individual manuals packaged with BEYOND® whitening kits.

PRE-TREATMENT

Perform a pre-treatment oral exam and consult with the patient on his or her desired whitening results to determine the best treatment plan for his or her individual situation. Ask the patient to sign the consent form and keep a copy for your records. Determine the pre-treatment color of the patient's teeth. (We recommend using the VITA Bleached guide 3D-MASTER® shade guide.) Take a photo to document the shade before starting treatment (optional). Turn on the BEYOND® II Whitening Accelerator and program the desired treatment time and light intensity as discussed with the patient.

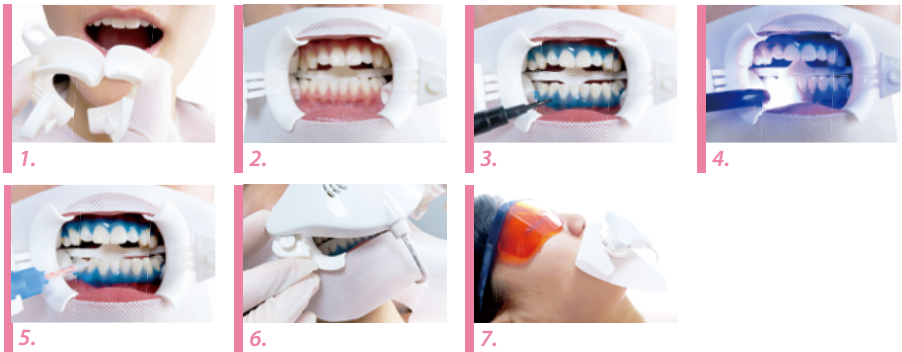

1. Ask the patient to take a seat in the treatment chair and recline the seat back to a 40° angle.
2. Place the protective eye goggles on the patient.
3. Use polishing sand to polish and clean the teeth before the treatment. Add water as necessary.
4. Ask the patient to rinse his or her mouth.
5. Apply lip protection cream and insert the BEYOND® II Cheek Retractor with the tongue guard pointing down.(Fig. 1)
6. Stretch the face protection cloth around the outer rims of the cheek retractor and fold the top down under the arms of the cheek retractor.
7. Insert cotton rolls on either side of the buccal frenum.(Fig. 2)

8. Use compressed air to dry the surface of the teeth and gingiva. Holding the syringe at a 45° angle to the tooth surface along the gingival margin, apply a 2-3 mm layer of BEYOND® BlueSeal® Gingival Protection along the gingival margin.(Fig. 3)
9. Apply extra lip protection along the inside of the upper and lower lips.
10. Use an LED Curing Light to cure the BEYOND® BlueSeal® Gingival Protection. Hold the LED Curing Light 0.5-1 cm away from the gingival. Move the device in a circular motion over the gingival area and light cure the gingival protection for 20-30 seconds (about 2-3 seconds per tooth). Use a dental probe tool to check that the gingival protection is fully cured. DO NOT over-cure the gingival protection.(Fig. 4) See pg.11-12 for more information on how to use the LED Curing Light.
11. Remove the whitening gel syringe and tip from the vacuum-sealed foil wrapper. Attach the mixing tip to the dual barrel syringe of whitening gel.
12. Apply a 2-3 mm layer of BEYOND® Whitening Gel to the dry surface of the teeth.(Fig. 5)
13. Attach the BEYOND® II LED light to the BEYOND® II cheek retractor and connect the two using the brackets on the retractor.(Fig. 6)
14. Once the LED light is attached, press the START  button to begin the first cycle.(Fig. 7)
15. Remind the patient that it is safe to swallow his or her saliva during the treatment.
16. Provide the patient with music or entertainment during the treatment.
17. At the end of the first cycle, the light will turn off automatically. Use suction to remove the whitening gel. DO NOT rinse with water. Repeat the gel and light application steps two more times for a total of three treatment cycles.
18. After the final cycle, suction off the whitening gel, remove the gingival protection, cotton rolls, face protectioncloth, cheek retractor, and protective eye goggles and ask the patient to rinse his or her mouth with water.

POST-TREATMENT

Show the patient his or her new teeth color and give him or her a Post-Whitening Care Instruction Card. Remind the patient that for the next 24 hours, he or she should avoid colored food and drinks.

Use of tobacco products should also be limited. Inform the patient that, depending on his or her diet, oral hygiene, and use of tobacco products, periodic touch-up treatments may be necessary to maintain his or her whitening results.


The BEYOND® II Whitening Treatment is a straightforward and simple procedure. If you have additional questions regarding the treatment after reading this system manual, please do not hesitate to email BEYOND® at info@beyonddent.com.

AIR IONIZER PURIFIER



The BEYOND® II Whitening Accelerator comes equipped with an air ionizer that silently cleans and sanitizes air in areas larger than 6000/cm³.

The ionizer is designed to remove airborne pollutants, allergens, pollen, mold, smoke, unpleasant odors, dust and other airborne pollutants.

The Ionizer function is able to be operated once the POWER ON/OFF button is ON. Then, The Ionizer function may be turned on by pressing the  button on the screen.

Whitening Gel Storage and Maintenance

BEYOND® Whitening Gel syringes must be stored at temperatures near 5°C. Do not use the gel if the following are present: Foil package around whitening gel syringes is no longer vacuum-sealed, there is gel leakage evident, the gel is no longer viscous and is in a liquid state, or the syringe plunger is being pushed out of the syringe body. If your whitening gel has started to break down, please contact your local dealer or BEYOND® directly for replacements.



Please avoid your eyes to direct exposure to the LED light.

In order to ensure the normal use of products and the safety of the electromagnetic interference, please avoid putting BEYOND® II Whitening Accelerator in the strong electromagnetic interference environment or near an interference source.

CLEANING THE DEVICE

The BEYOND® II Whitening Accelerator should be cleaned and disinfected after each whitening session. Spray disinfectant spray onto a clean cloth and wipe clean the BEYOND® II LED Light.

A slight temperature change of the BEYOND® LED Light is normal if using the high-power light source.

Safety Measures

Teeth whitening is not suitable for pregnant or lactating women or children under 16 years of age. BEYOND® is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.

Only allow trained personnel to operate the system and provide the whitening procedure.

Troubleshooting

If BEYOND® II Whitening Accelerator does not turn on.

1. Check that the power cord is firmly inserted into the power outlet and the machine base.
2. Check that the SYSTEM ON/OFF switch at the base of the machine is turned on.

Manufacturer's Contact Information

BEYOND Dental & Health USA Headquarters

12503 Exchange Dr. Suite 530
Stafford, TX 77477 USA
TEL +1-281-277-4352
FAX +1-281-277-4350
E-mail: info@beyonddent.com

BEYOND Dental & Health European Headquarters

Melkerstr. 4
14167 Berlin, Germany
Tel: +49-304-908-49-57
E-mail: europe@beyonddent.com

BEYOND Dental & Health Asia Pacific Headquarters

Chaoyang District, Beijing
China 100029
E-mail: asia@beyonddent.com










BEYOND Latin America Headquarters

10760 NW 123 St.
Medley, FL 33178
Tel: +1-954-554-6960
E-mail: latam@beyonddent.com

European Representative Office

MDSS GmbH
30175 Hannover
Germany

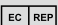

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
	Consult Instructions for Use		Use By		Date of Manufacture
	Caution		Batch Code		Temperature limit
	Authorized Representative in the European Union		Manufacturer		Do Not Reuse